

URIGE™

MULTI-PARAMETER URINALYSIS TEST STRIPS



URIGE™ Test Strips are ready to use, straight out of the bottle. No additional components or laboratory equipment necessary.

URINALYSIS



- > **EASY TO USE**
Just dip and read
- > **RAPID**
Test results in seconds
- > **FLEXIBLE**
Multiple formats allow for simultaneous determination of up to 10 parameters

URIGE™ TEST STRIPS provide a wide range of nearly-instantaneous information on the status of carbohydrate metabolism, kidney and liver function, acid-base balance, and bacteriuria.

ORDERING INFORMATION:

Product Description	Catalog No.	Format
Urige - 1 Strip Test	20010-100	100 tests/bottle
Urige - 2 Strip Test	20020-100	100 tests/bottle
Urige - 3 Strip Test	20030-100	100 tests/bottle
Urige - 4 Strip Test	20040-100	100 tests/bottle
Urige - 5 Strip Test	20050-100	100 tests/bottle
Urige - 6 Strip Test	20060-100	100 tests/bottle
Urige - 7 Strip Test	20070-100	100 tests/bottle
Urige - 8 Strip Test	20080-100	100 tests/bottle
Urige - 9 Strip Test	20090-100	100 tests/bottle
Urige-10 Strip Test	20100-100	100 tests/bottle
Urige-11 Strip Test**	20110-100	100 tests/bottle

Test Parameters

- Glucose
- pH
- Protein
- Ketone
- Blood
- Bilirubin
- Nitrite
- Urobilinogen
- Specific Gravity
- Leukocytes
- Ascorbic Acid**
- Microalbumin Compensation Area**

*CLIA-Waived approval pending. **Available upon request.


URIGE™



MULTI-PARAMETER URINALYSIS TEST STRIPS

Test Procedure

1 Place strip in patient urine sample.

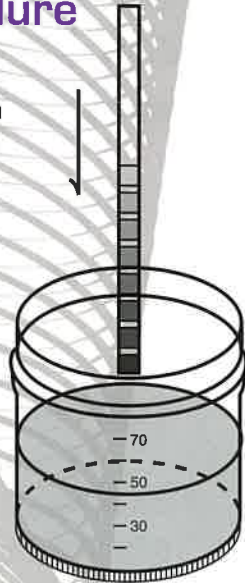

2 Wait 30 - 120 Seconds.



3 Read results by comparing reagent areas to corresponding color key squares on the bottle label.

LEUKOCYTES 2 min	NEGATIVE 0	TRACE 15	SMALL (+) 75	MODERATE (++) 125	LARGE (+++) 500 CELLS/μL		
NITRITE 60 sec	NEGATIVE	POSITIVE (ANY DEGREE OF PINK COLOR)					
URO-BILIRUBIN 60 sec	NEGATIVE	1	2	4	6		
PROTEIN 60 sec	NEGATIVE	TRACE	30 (+) 0.3	100 (++) 1.0	300 (+++) 3.0	1000 (++++) 10.0	
PH 60 sec	5.0	6.0	6.5	7.0	7.5	8.0	8.5
BLOOD 60 sec	NEGATIVE 0	NON-HEMOLYZED 10 TRACES	HEMOLYZED TRACE	SMALL (+) 25	MODERATE (++) 50	LARGE (+++) 200 CELLS/μL	
SPECIFIC GRAVITY 45 sec	1.000	1.005	1.010	1.015	1.020	1.025	1.030
KETONE 40 sec	NEGATIVE	TRACE (5) 0.5	SMALL (15) 1.5	MODERATE (40) 4.0	LARGE (80) 8.0	LARGE (160) 16.0	
BILIRUBIN 30 sec	NEGATIVE	SMALL	MODERATE	LARGE			
GLUCOSE 30 sec	NEGATIVE	100	250 (+) 15	500 (++) 30	1000 (+++) 100	2000 (++++) 1000	3000 (+++++) 1500

TESTS AND READING TIME

IMPORTANT! READ PACKAGE INSERT BEFORE USE.
DO NOT touch test areas. Store at temperatures between 15°-30° (59°-86° F). DO NOT remove desiccants. Be sure to replace cap immediately and tightly. V/B

> A Winning Combination!

Results with Urige™ Test Strips are obtained in clinically-meaningful units directly from the Color Chart comparison. The color blocks represent nominal values; actual values will vary around the nominal values.



SYNTRON
Bioresearch, Inc.

2774 Loker Ave. West | Carlsbad, CA 92010
Tel: (760) 930-2200 | Fax: (760) 930-2212 | E-mail: marketing@syntron.net
For a list of all Syntron Bioresearch, Inc. products, visit our website: www.syntron.net

Contact Syntron Bioresearch, Inc. for more information about our Urige™ products. Call, fax, or e-mail us your order. Urige™ is a registered trademark of Syntron Bioresearch, Inc. All rights reserved. *CLIA-Waived approval pending.

Doc ID: PDS-060
Revision: 02
Eff. Date: 03-31-09