

# Absoludy COVID-19 IgM/ IgG Rapid

Currently available under an FDA EUA & CE-IVD & MFDS

## FEATURES

### FAST

Test result in 15 minutes

### ACCURATE

Sensitivity: 90.24%, Specificity: 100%

### EASY TO USE

No special knowledge required  
Use 10 µL of serum, plasma or whole blood



Absoludy COVID-19 IgM/IgG Rapid is based on the principle of an immune-chromatography in vitro test for the qualitative determination of SARS-CoV-2 virus (COVID-19) specific IgM or IgG in serum, plasma, or whole blood. Absoludy COVID-19 IgM/ IgG Rapid gives clinical decision-making for patients suspected of SARS-CoV-2 virus (COVID-19) infection.

## PERFORMANCE

< Total Results for IgM & IgG Clinical Agreement >

Method		Allplex™ 2019-nCov Assay		
		Positive	Negative	Total
Absoludy COVID-19 IgM/IgG Rapid	Positive	37	0	37
	Negative	4	92	96
Total		41	92	133


Clinical Sensitivity: 90.24%

Clinical Specificity: 100%

## PROCEDURE


### ① Take Sample

Dip the sample collection tube into the specimen (serum, plasma or whole blood) and take the specimen (10 $\mu$ l)

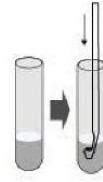

### ② Load Specimen

Transfer 10 $\mu$ l of sample (serum, plasma or whole blood) to sample well using specimen collection tube or micropipette

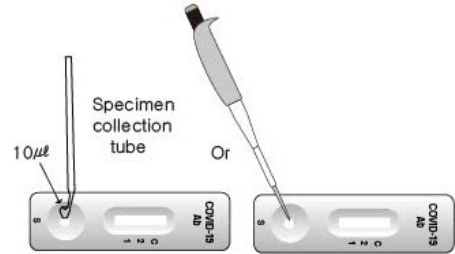

### ③ Load Sample Diluent

Add 3 drops (about 100 $\mu$ l) of sample diluent into the sample input well of the test device.



### ④ Read & Interpret

Read the test result within 10~15 minutes.  
Do not read the test result after 15 minutes.

## RESULTS

### Negative



### Positive



COVID-19 IgG positive



COVID-19 IgM positive



COVID-19 IgG/IgM positive

### Invalid



## ORDER INFO.

Model No.	RDT-COVab01+	RDT-COVab05+	RDT-COVab25+	RDT-COVab25
Product class	Rapid Kit			
Decription	Absludy COVID-19 IgM/IgG Rapid			
Contents	Test Devices_1test Sample collection tubes_1ea Sample diluent_0.5mL Instructions for use_1 ea <i>Disposable lancet_1ea</i> <i>Alcohol swab_1ea</i>	Test Devices_5tests Sample collection tubes_5ea Sample diluent_1mL Instructions for use_1 ea <i>Disposable lancet_5ea</i> <i>Alcohol swab_5ea</i>	Test Devices_25tests Sample collection tubes_25ea Sample diluent_4mL Instructions for use_1 ea <i>Disposable lancet_25ea</i> <i>Alcohol swab_25ea</i>	Test Devices_25tests Sample collection tubes_25ea Sample diluent_4mL Instructions for use_1 ea

# ABSOLGY

#1303, Digital Empire "B" Simin-daero 383, Dongan-Gu,  
Anyang-Si, Gyeonggi-Do 14057 REPUBLIC OF KOREA  
TEL. 82-31-348-8637 / FAX. 82-31-348-8635  
E-mail. sales@absology.co.kr / www.absology.co.kr

