


## DISINFECTANTS

# VIRUSOLVE+ RTU MEC TEST STRIPS

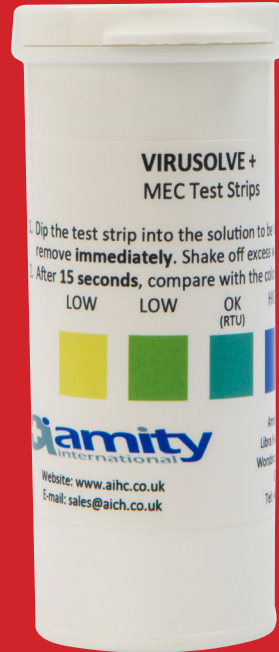

1. Dip test strip into soaking bath  
Virusolve+ RTU solution.



2. Shake off excess liquid and wait  
15 seconds.



3. Compare colour to colour chart



Amity Limited, Libra House, West Street,  
Worsbrough Dale, Barnsley, South Yorkshire,  
S70 5PG, UK

T: +44 (0) 1226 770787

F: +44 (0) 1226 770757

E: sales@amityinternational.com

[WWW.AIHC.CO.UK](http://WWW.AIHC.CO.UK)

© Amity Limited all rights reserved 2018