

**How to apply**

\*Use with caution to avoid irritating to the scar★.

**1** Partially remove the white paper backing (larger piece).

**2** Return the tape to the front and pull the release liner on the left and right with both hands.

**3** Align the central part with the scar and apply firmly while pulling.

**4** Remove the white paper backing and apply the entire tape, pressing firmly from above.

**5** Remove the liner and smooth the tape against the skin.

For M size, the application method is different. Please see How to apply in the user instructions carefully prior to use.

**Exclusive tape for scar care**

Uses: To protect the surface layer of the scar★ after suture removal



**ATLESS-CARE specs**

Product name	No.	Size	Product Code	Individual package		Outer package	
				Contents	JAN [4971620]	Case	GTIN [14971620]
ATLESS-CARE	No.S10	25mmx50mm	YB-Q2550S1	10 sheets	892077	24 boxes	892074
	No.S30	25mmx50mm	YB-Q2550S	30 sheets	892053	24 boxes	892050
	No.M10	50mmx50mm	YB-Q5050S1	10 sheets	892060	24 boxes	892067

\* Suggested retail price excludes tax.

[Notes on Use] ● ATLESS-CARE is a non-sterilized tape used for scar care\* after suture removal. Do not apply directly to wound sites. ● To ensure safety, be sure to follow the doctor's directions. ● If rashes, flaring, itching, blisters, or other symptoms arise, discontinue use and consult a physician or pharmacist. ● Clean the area to which you plan to apply ATLESS-CARE by wiping off moisture and other contaminants. Moisture may impair adhesion. ● Use with caution when applying to the face and head. The tape may come off when applied to oily areas. ● Apply carefully to avoid peeling or lifting. Reapply the tape if you notice peeling or lifting. \*Apply while stretching the skin when applying to areas around joints. (Follow any specific instructions given by your doctor.) \*Use with caution on subjects prone to develop rashes with adhesive bandages. ● ATLESS-CARE is an elastic product. However, excessive pulling may cause the tape to break or tear. Handle with care. \*Use the number of pieces appropriate for the length of the scar.

[Storage] Store in a cool, dry place out of direct sunlight. ● Keep out of reach of children.

[Standard for tape change] Change the tape once every two or three days. Peeling indicates the tape should be changed.

[Note on scar\*] Scar is assumed to be a closure site without exudate or bleeding.

[Note on "Zero Feel"] "ZERO Feel" describes the feel of having not tape on the skin.

**skinix®**  
 KYOWA LIMITED    www.skinix.jp  
 Osaka head office: 20-28 Tachibana 3-chome,  
 Nishinari-ku, Osaka-shi, Osaka  
 557-0051, Japan

For inquiries or information  
 Tel: +81-6-6658-8218  
 Fax: +81-6-6658-8101

ZZ009 18.08-1,000 No.1

Exclusive tape for scar care

**ATLESS-CARE**

A new and proprietary tape for scar care after suture removal

# Advent of ATLESS-CARE

“The wound is closed, but...” Concerns or worry don’t always fade after the wound is closed.

It’s led to the advent of ATLESS-CARE, a new proprietary tape for scar care.

We developed our independent pinpoint care structure by using the restoring force of the tape and realized comfortable “ZERO Feel” feel when using an ultrathin, 7-micron film. \*

These efforts have led to dramatic improvements in scar care.

**ATLESS-CARE is a revolutionary proprietary tape for scar care based on advanced scar care technologies.**

※We undertook a study on minimizing surgical scars through wound protection in cooperation with Dr. Kotaro Yoshimura in the Department of Orthopedic Surgery, the University of Tokyo School of Medicine Hospital (now at Jichi Medical University Hospital).


## The shrinking force of the stretched tape strengthens the approaching force of skins to the scar★.

### Care with ATLESS-CARE

Relies on the restorative force of elastic tape  
Only the central part of the tape stretched by pulling shrinkage, strengthening the approaching force of the tape to the skin.



Stretch the central part of the tape by pulling with both hands and apply the stretched tape to the scar. The stretched tape seeks to return to its original shape and contracts.

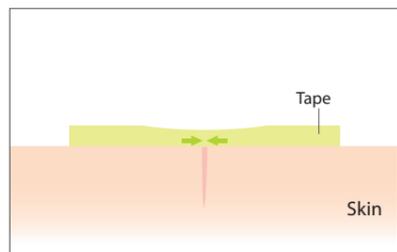


Only the central part of the tape applied to the skin shrinks. This enables adding the approaching force to the expanding skin.

### Care with ordinary surgical tape

A firm tape that lacks elasticity does not shrink, so it is unable to add the approaching force to the skin. (\*Restoring)



Surgical tape is applied to cover the scar.

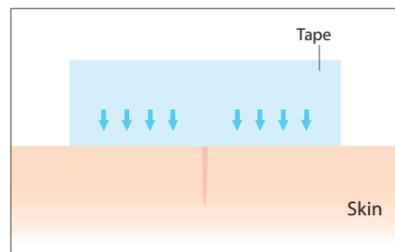

The tape secures the skin by pressure from above. This is unable to add the approaching force to the expanding skin. (compared to similar Kyowa products)

Comparing to ordinary surgical tape, it resists expanding the scar.  
**ATLESS-CARE accomplishes a revolutionary scar care solution by adding restoring force to skin.**


### Feature 1

## Pinpoint care structure that maximizes the restorative power of the tape

Our one-of-a-kind structure makes the central part of the tape uniformly elastic (\*under patent application), and it allows the approaching force to be uniformly added to the skin where scar is present.


#### With ATLESS-CARE

- ① Because a strong film is present above the main tape, uniform force is applied when pulling the tape.
- ② Only the film in the central part of the tape is removed by pulling the tape, thereby extending the tape accurately.
- ③ The tape is shrunk in a pinpoint manner by applying the central part of the tape to the scar. Thus, the approaching force that draws the tape to the scar is uniformly added.

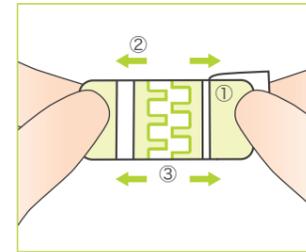

#### With ordinary film dressing

- ① Because the film on the main tape is not strong enough, force strength differs depending on the pulling force of the tape.
- ② The tape and film peel off immediately after pulling the tape. Stretching condition is sparse depending on the pulling force of the tape.
- ③ The tape sparsely shrinks when applying to the scar. Thus, the force that draws the tape to the scar does not function uniformly.

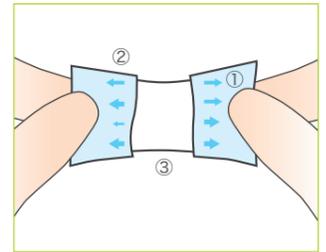

(Compared to similar Kyowa product)


### Feature 2

## Always comfort when apply the tape

ATLESS-CARE provides more comfortable, scar care for long time by reducing skin stress such as sticky and sweaty feel (minimize stress for skin) and overstressed skin.


#### A sense of “ZERO Feel” feel\* with an ultrathin, 7-micron film

Only 7-micron thickness film softly fits the skin texture. Due to very natural application feeling, you may forget it is on skin. Because vapor permeability is extremely high, and thus, evaporation of perspiration is not impeded, the patient experiences little itchiness or unpleasant feeling from moisture retention even when applying to the skin for extended periods.

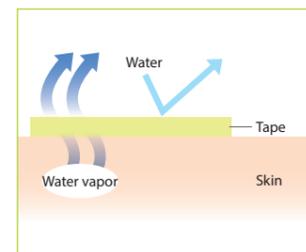

#### Pinpoint treatment does not make overstress for skin

Because ATLESS-CARE is advantageous in contracting the central part of the tape (the scar to be applied) alone in a pinpoint manner, it softly fits the surrounding area without overstress. Its excellent elasticity enables this film stretch or shrink smoothly along the motion.

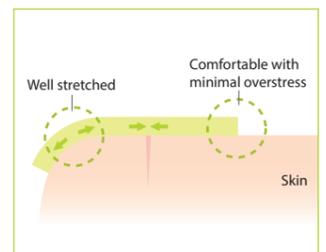


### Feature 3

## Professional treatment by only application just applying ATLESS-CARE.

Simple procedure with the patient’s hands. Scar care is possible at home.

By only stretching centre of ATLESS-CARE to the scar by stretching the central part of the tape.  
Scar care can be achieved by using the restoring force of the tape itself.  
We focused on the tape structure and procedure that allow patients themselves to easily apply the tape. Tape change is also simple. The patient can continue scar care without stress for extended periods.  
\*ATLESS-CARE can be applied to the scar that is completely closed without exudate immediately after suture removal.

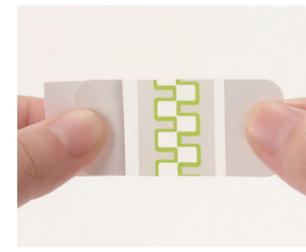

Simply apply the central part of the tape to the scar after pulling and stretching the tape by using uneven lines at the central part of the tape as an indicator.

#### •Scar self-care by the patient takes long time.

##### Physicians and Nurses

Explain the method of procedure (how to apply) to patients.

##### Patients

Buy ATLESS-CARE at a hospital store, etc.

Change the tape once every two and three days.

Scar treatment procedure, which has been given up by the patient, is greatly improved, and the patient’s anxiety and stress are thus reduced.

The first proprietary tape in the healthcare industry, which makes a tremendous improvement in scar care

<ATLESS-CARE>