

SilkLance™—Good choice for home care

Product Features

Safety

Gamma irradiation ensures 5 years of shelf life / Needle tip is sealed in the protective cap / Needle tip automatically retracts after use

Low pain level

US&CN patented low pain level design

Mini appearance




The smallest safety lancet in the market, elegant appearance and eco-friendly, easy to use and carry

Various gauges

21G, 23G, 26G, 28G, 30G, various gauges, meet different kind of skin type



Smallest safety lancet

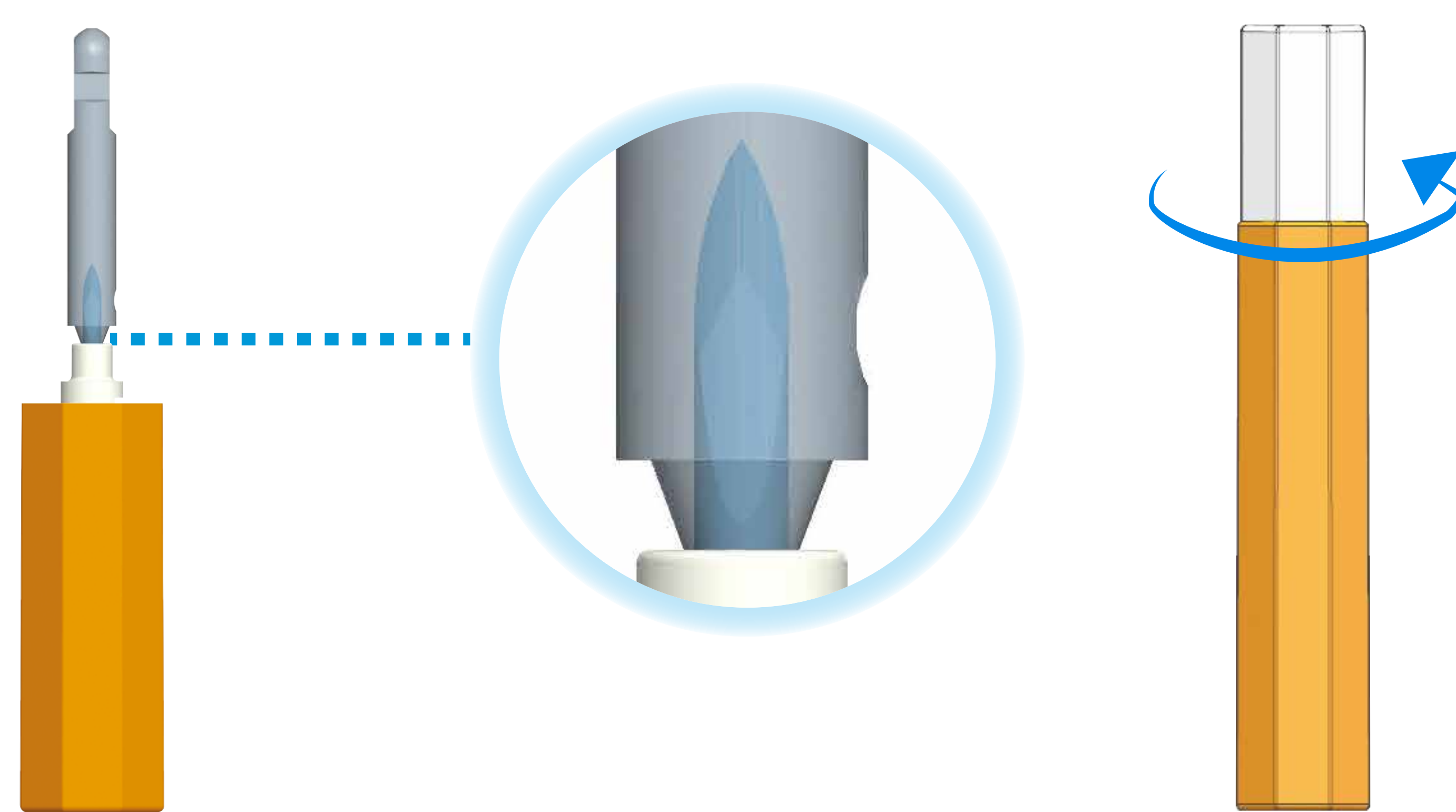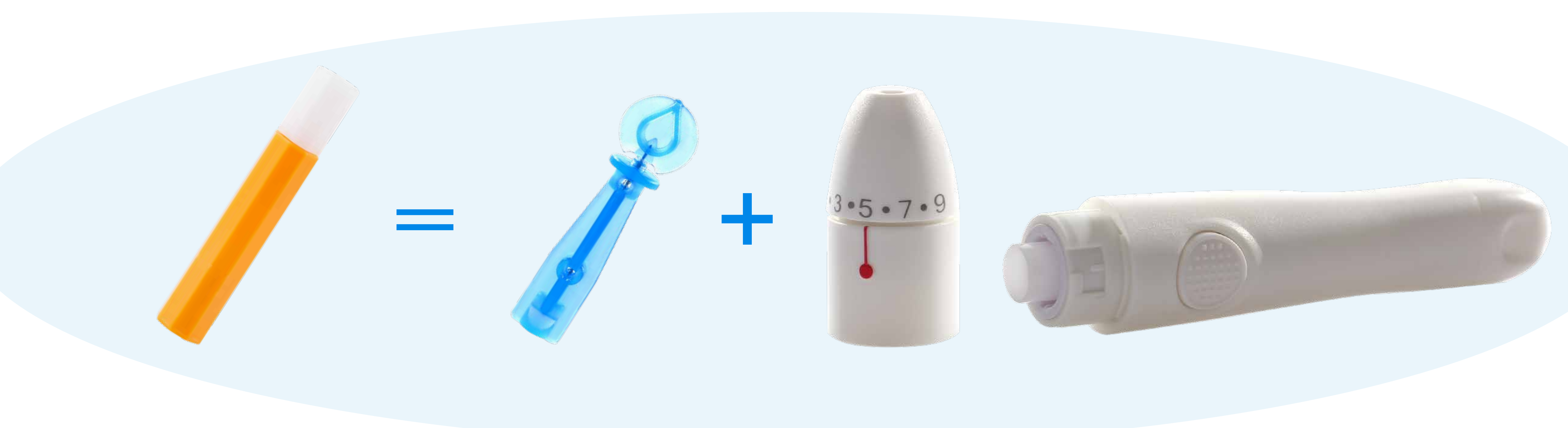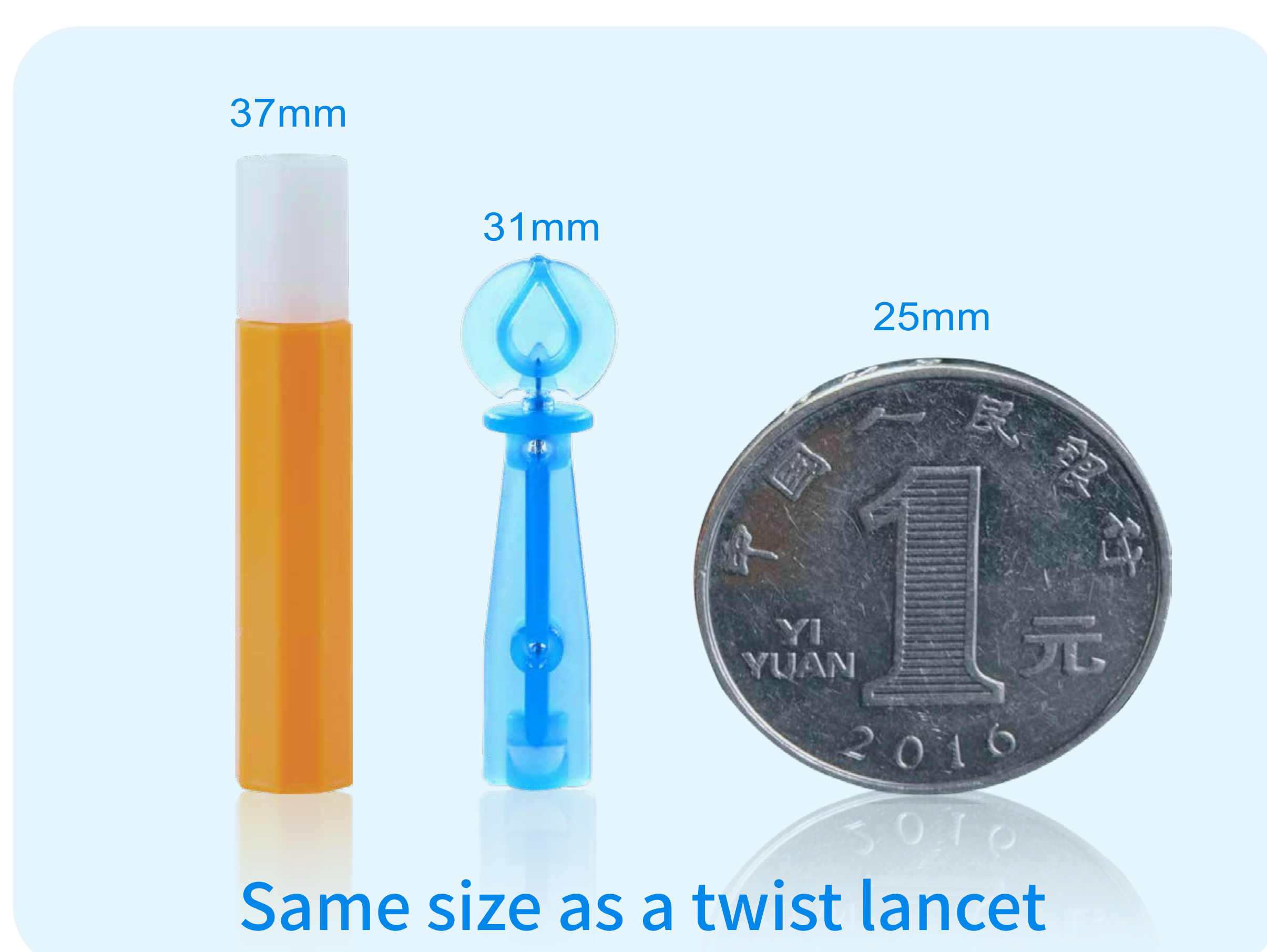

Needle tip is sealed in the plastic handle, ensuring the sterility
Twist off the protective cap, do not pull



US&CN patented design, stable firing system




Invention Patent

US: 9.826.927


CN: ZL 2012 1 0080546.3

- Stable firing system
- Elastic inner sleeve
- Actual-location puncture
- No repeated puncture


Use Instructions for MejorLance

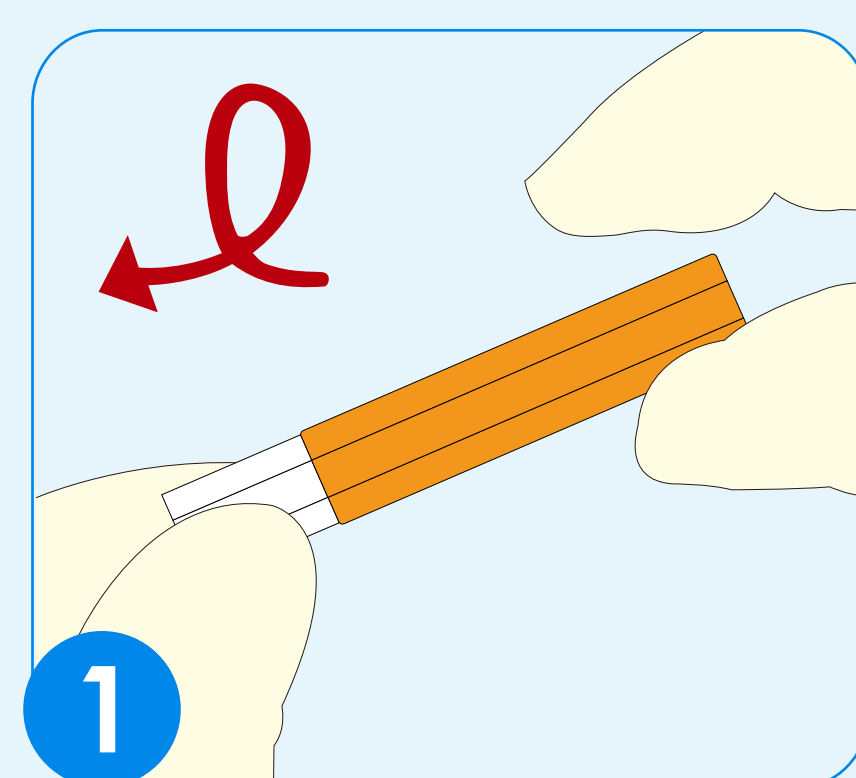

1. Twist and pull the protective cap off. Discard it into the container.

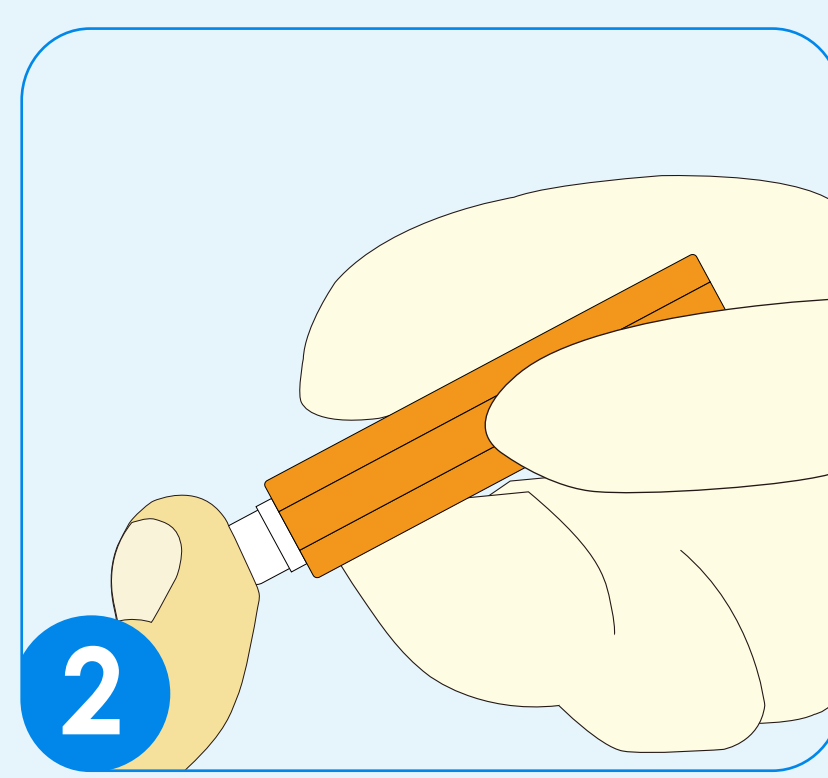

2. Press the lancet against the blood sampling site to get the sample.

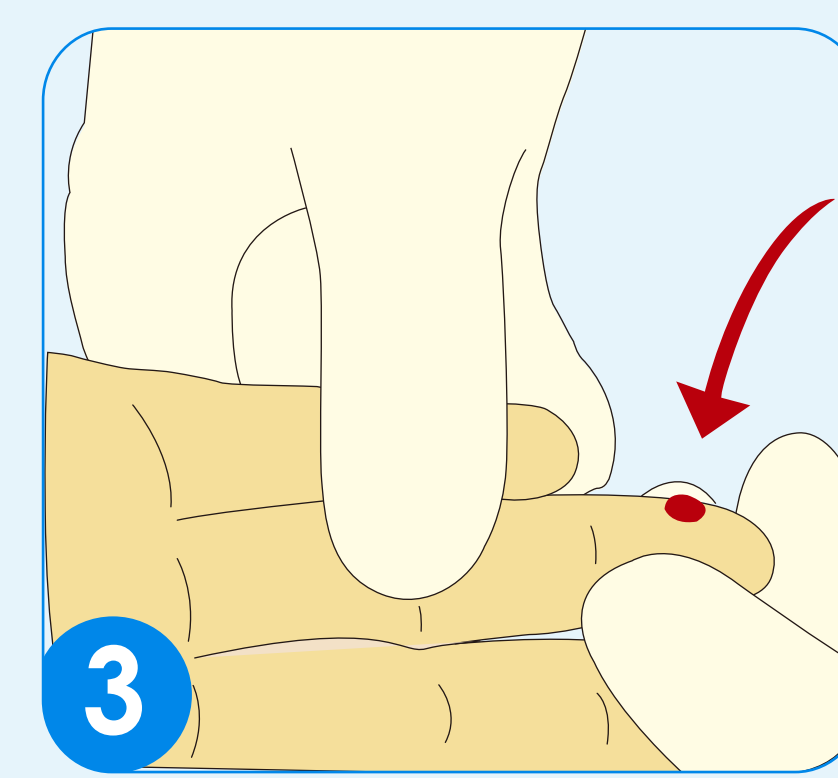

3. Massage the sample site gently to get the sample, and discard the used lancet into a medical waste container.

Note: Please dispose the lancet into appropriate container at once after using.

Code	Specification	Color	Blood	Package
I22A2118100	21G*1.8mm			100 pcs/box 40 boxes/carton (or per customers' requests)
I22A2318100	23G*1.8mm			
I22A2616100	26G*1.6mm			
I22A2816100	28G*1.6mm			
I22A3014100	30G*1.4mm			