




# Sim Limb

Patent Pending

**Blood Squeeze Bottle**

**Wound Packing**

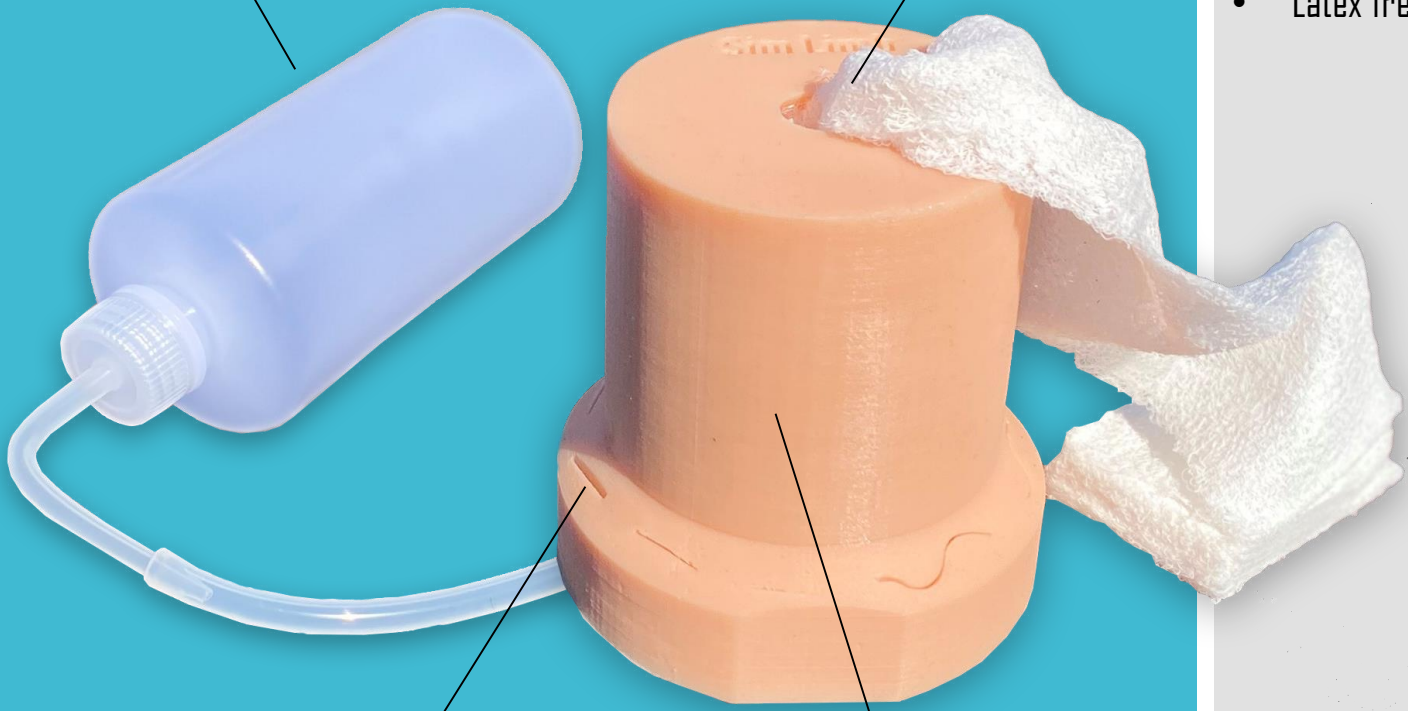

**Suture Cut**

**Tourniquet Trainer**

## Features

- Wound packing
- Tourniquet Trainer
- Suture cuts
- Chest seal trainer
- Latex free

- Sim Limb is a great way to help cover the basic trauma care techniques.
- It's a must for any Stop the Bleed class training students the principles of tourniquet application, wound packing, and chest seal application.

## Contents

- Sim Limb
- Carry Case
- Squeeze Bottle
- Compressed Gauze

\*Tourniquet not included

# Sim Limb Applications

## Tourniquet & Bandage


Proper tourniquet application can be practiced by wrapping and tightening the device around the Sim Limb. This will help train proper tourniquet application to an extremity by making sure the tourniquet is tight enough around the extremity and all steps of the tourniquet application have been accomplished. **Please follow the manufacturer guidelines and instructions for the model tourniquet that is in your possession.**

## Wound Packing

Sim Limb has a unique injury sight that can be utilized to wound pack. The trainer has a simulated artery in the wound. This portion of the simulator can actively bleed with the provided simulated blood and blood squeeze bottle, to give a real life like scenario of stopping the bleed.

## Chest Seal

The same section for wound packing can be used to practice chest seal application. The wound simulates an entry wound that needs to be sealed using a chest seal. **For proper chest seal applications, please follow the manufactures guided instructions for the model chest seal that you possess.**

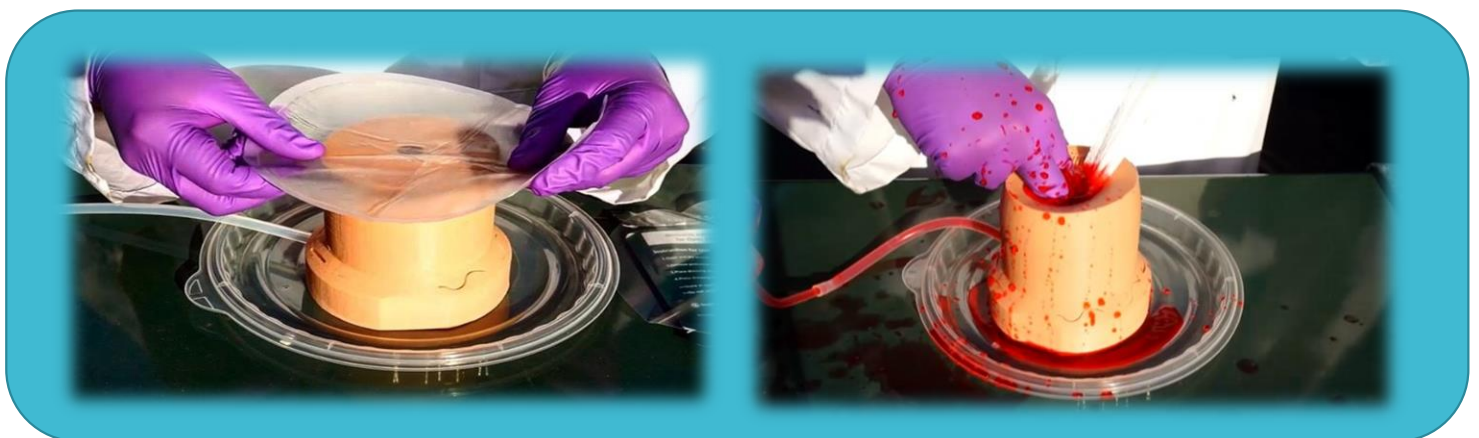

\*For more information, please visit our website.  
Youtube channel: **Novara Solutions**