


I.V. Pole

2 & 4 hook mobile

MedStor[®]
The heart of your storage solutions

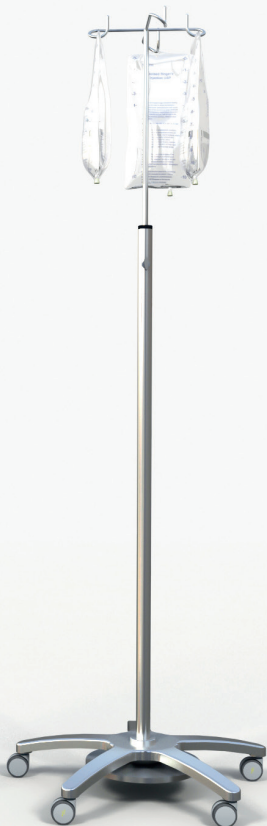

I.V. POLE

Secure and stable stainless-steel design

I.V. poles are essential equipment wherever an I.V. needs to be administered. They are designed to hold a bag of fluid or medicine that is being delivered intravenously to hospital patients at a steady flow.

The Medstor stainless steel I.V. pole comes with a choice of two and four hook configurations to suit the needs of the clinical team.

The fully mobile base enables the I.V. pole to be taken directly to hospital wards and treatment rooms, allowing it to be quickly in use at the point of care.

It also has a simple and easily deployed handwheel height adjustment, so can be swiftly set to the height required.

Robust stainless steel construction makes the I.V. pole easy to clean, in line with hospital infection prevention and control practices.

Conform to the European Directives

Medical Device Directive 92/42/EEC-Class 1
BS3619-1976 Stability Test.


Code

Description

MS33020000	Stainless Steel 2 Hook IV Pole / Drip Stand
MS33020001	Stainless Steel 2 Hook IV Pole / Drip Stand with Weighted Base
MS33040000	Stainless Steel 4 Hook IV Pole / Drip Stand
MS33040001	Stainless Steel 4 Hook IV Pole / Drip Stand with Weighted Base

WARNING

- Only to be used on a flat surface with an inclination no greater than 9°
- Take caution when walking with I.V. Pole
- Please ensure the Medstor unique serial number is not removed

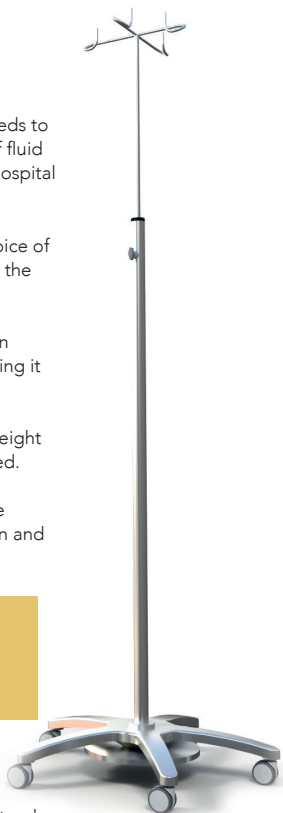

I.V. POLE

Features

- ① 2 & 4 hook 12mm stainless-steel rod
- ② Handwheel height adjustment
- ③ 5 star polished base
- ④ 3 x 50mm nylon castor
- ⑤ 2 x 50mm anti-static nylon castor
- ⑥ 32mm DIA pole with secure end caps
- ⑦ Optional weighted base

Height range
1400 to 2200mm

BS3619 tested

The Design has been tested against the BS3619 Mobile Infusion Stand Stability Test (Clause 10).


Weight loading

Please ensure weight loading is evenly distributed for stability.

Max load 2kg per hook.

Cleaning Advice


I.V. pole can be cleaned with detergent and water. Do not use a scouring pad.

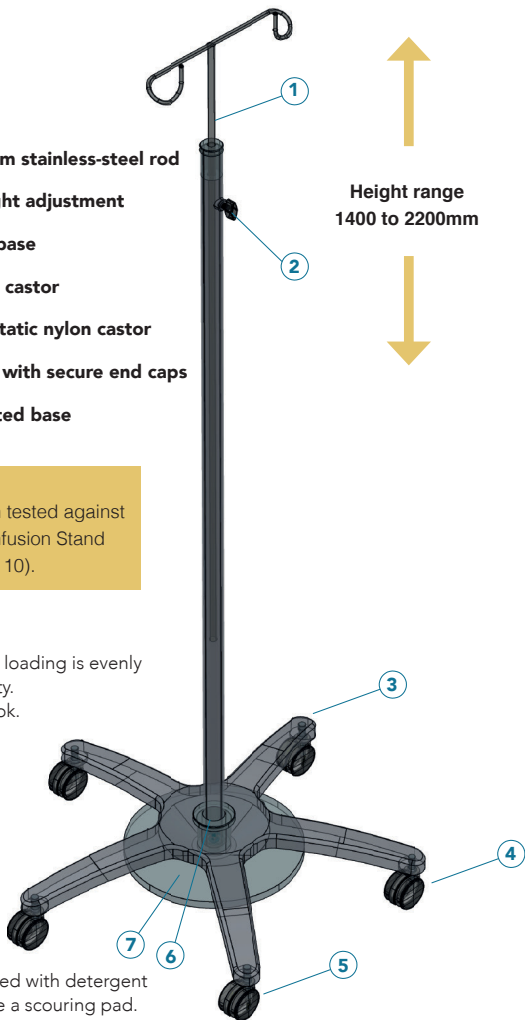

REGISTRATION DETAILS:



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DEALER INFORMATION:



UK HEAD OFFICE & SHOWROOM

Bayley Street, Stalybridge,
Cheshire,
SK15 1QQ

TEL: +44 (0)161 330 5577
EMAIL: health@medstor.co.uk

www.medstor.co.uk

IRELAND OFFICE

Unit 120, Grange Hill,
Baldoyle Industrial Estate,
Dublin 13.

TEL: +353 (0)1 839 3596
EMAIL: sales@medstor.ie

www.medstor.ie

