



2019-nCoV Antigen Rapid Test Kit

Colloidal Gold Assay

- **Short test time: 15 Minutes**
- **Accuracy: More than 90%**
- **Easy operation: Throat/Nose swab sample**
- **One-step solution**
- **Lower cost with high efficiency**

Add: 3rd floor, Xinhualian science and technology building, No.5 Jiachuang Road, Tongzhou District, Beijing, China

Tel: (8610) 69502385 Fax: (8610) 69507615 Whatsapp/Mob: +86-18242749295 (Jax)

Web: www.diagreat.com

Email: jax.chang@diagreat.com




Components

- ◆ 25pcs Nasopharyngeal swabs
- ◆ 25 test cassettes
- ◆ 25 Antigen extraction tubes and drippers
- ◆ 1 bottle antigen extract R1
- ◆ 1 Test tube stand



Application


- ◆ Suspected case test in epidemic area
- ◆ Early test of fever clinic and CDC
- ◆ Screening of primary medical institutions
- ◆ Inspection of floating population in public areas

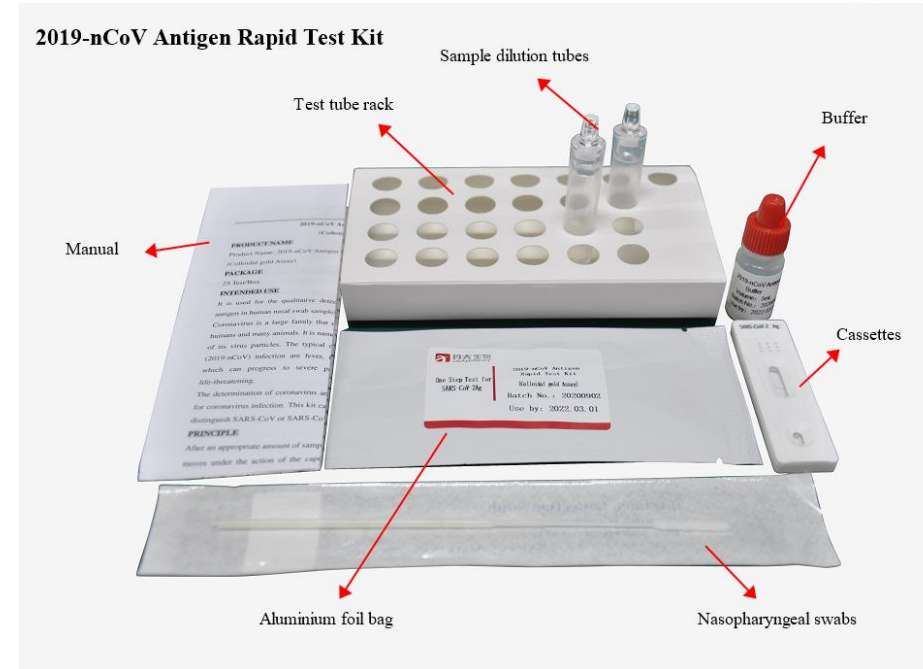

Product Information


| Product Name | Methodology | Size | Storage and Validity | Specimen |
|---|----------------------|--------------|-----------------------------------|----------------|
| 2019-nCoV Antigen Rapid Test Kit | Colloidal Gold Assay | 25 Tests/box | Stored at 2-30°C within 18 months | Nasopharyngeal |

Easy Operation

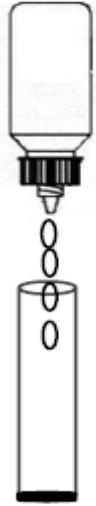

1. Add 5 drops (300 μ l) of R1 to the extractor tube.


6 times

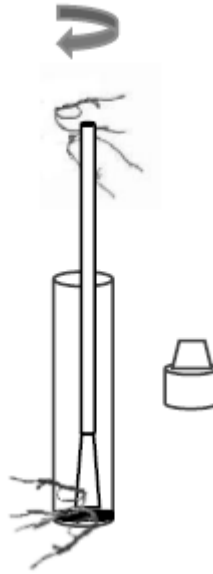


2. Insert the swab into the extractor tube, and rotate at least for 6 times in order to release the antigen from the swab.



3. Leave the swab in the tube for 60 seconds

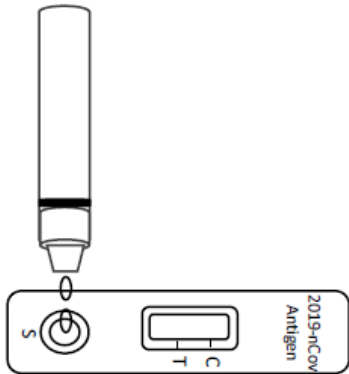

6. Add 2 drops (100 μ l) of the mixed sample to the cassette and read the results in 15 minutes

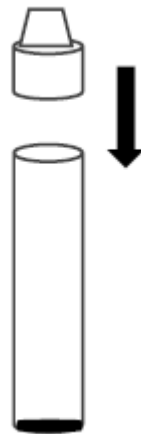


5. Put the tip on the tube and tighten up.



4. Squeeze the swabs at the tube wall to release remaining antigen liquid from the swab.

Test Result Interpretation

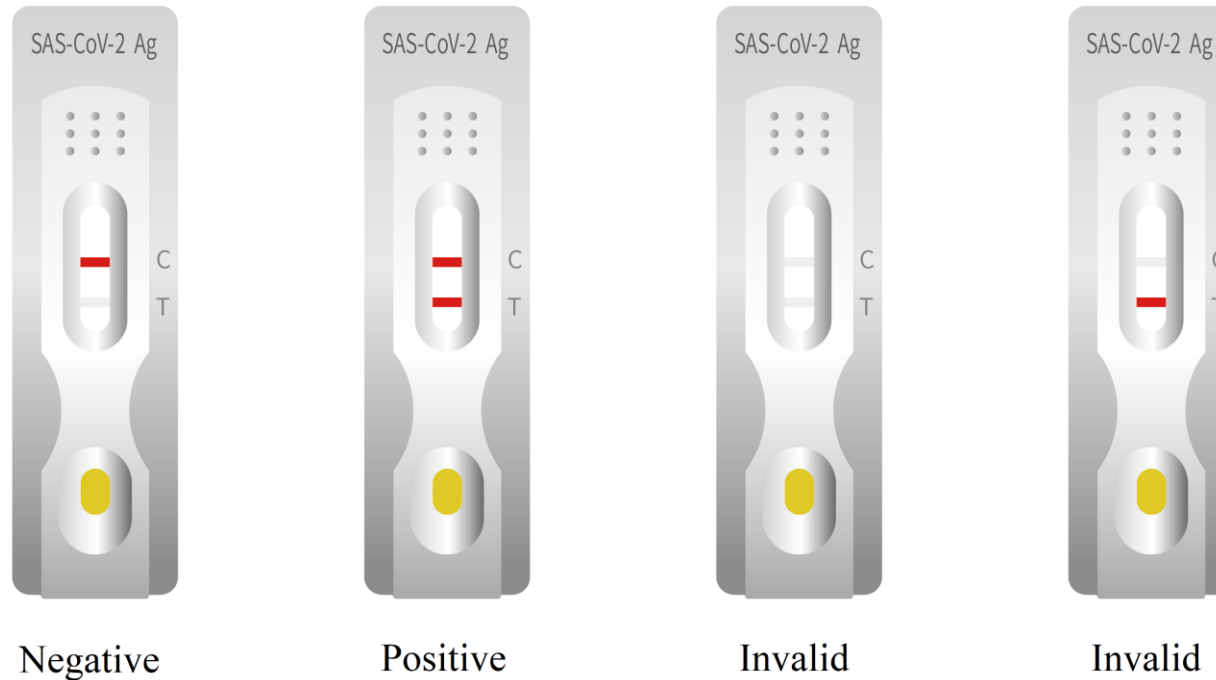

Read the results in 15 minutes. A strong positive results may show within 15 minute. But the negative result must be reported after 15 minutes. And the results after 20 minutes are not valid anymore.



THANKS

Add: 3rd floor, Xinhualian science and technology building, No.5

Jiachuang Road, Tongzhou District, Beijing, China

Tel: (8610) 69502385 Fax: (8610) 69507615

Whatsapp/Mob: +86-18242749295 (JAX)

Web: www.diagreat.com

Email: jax.chang@diagreat.com