



2019-nCoV Neutralizing Antibody Rapid Test Kit

Colloidal Gold Assay

- **Short test time: Within 15 Minutes**
- **Accuracy: More than 90%**
- **Easy operation: Whole blood, Serum and Plasma**
- **One-step solution**
- **Lower cost with high efficiency**

Add: 3rd floor, Xinhualian science and technology building, No.5 Jiachuang Road, Tongzhou District, Beijing, China

Tel: (8610) 69502385 Fax: (8610) 69507615 Whatsapp/Mob: +86-18242749295 (Jax)


Web: www.diagreat.com

Email: jax.chang@diagreat.com



Components

- ◆ 25 test cassettes
- ◆ 1 Test manual
- ◆ 1 bottle buffer
- ◆ 25pcs needle
- ◆ 25pcs disposable pipettors

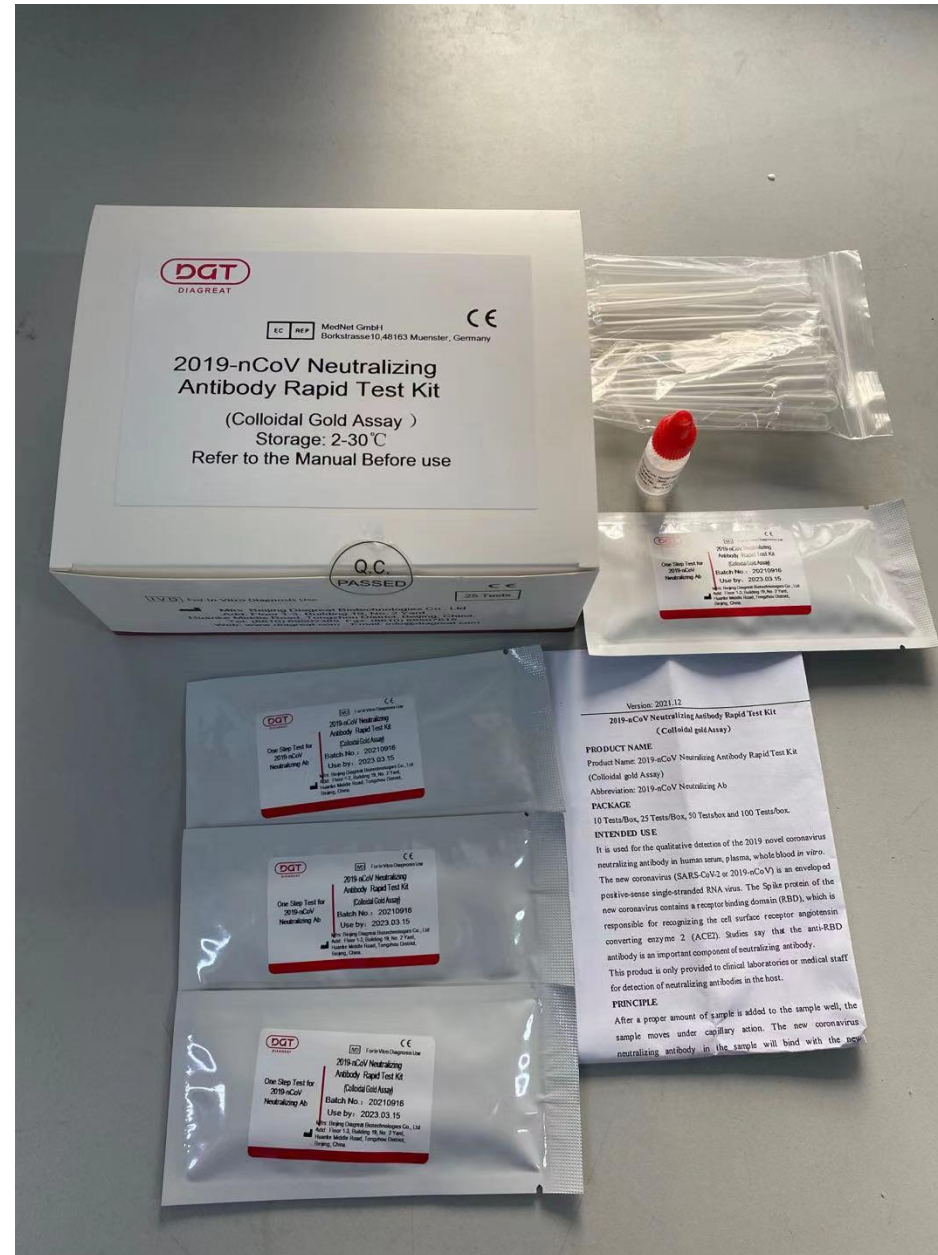


Application

- ◆ Evaluate immune system reaction after vaccination
- ◆ Advise the patient action during pandemic
- ◆ Evaluate population immune system protection

Product Information

Product Name	Methodology	Size	Storage and Validity	Specimen
2019-nCoV Neutralizing Antibody Rapid Test Kit	Colloidal Gold Assay	25 Tests/box	Stored at 2-30°C within 18 months	WB/S/P

Test procedure

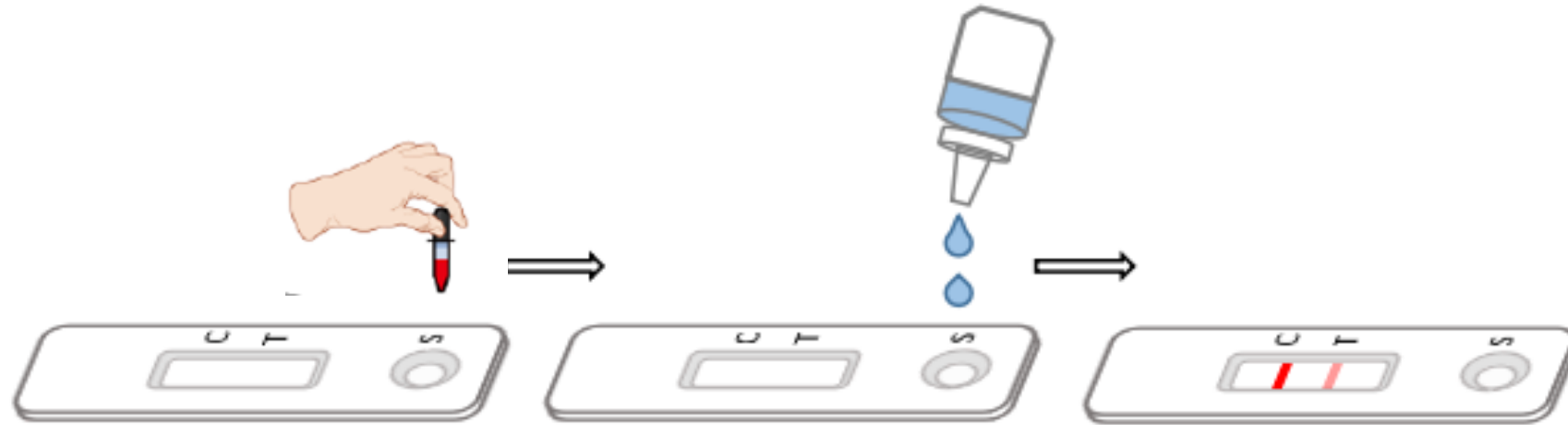

a. Add 10ul sample to cassette

b. Add 2 drops buffer to cassette

c. Read results in 15 minutes

Test result



Negative: Only C-line develops, the test indicates that no detectable 2019-nCoV Neutralizing Antibody is present in the specimen. The result is negative or non-reactive.



Positive: Both C and T lines develop, the test indicates the presence of detectable 2019-nCoV Neutralizing Antibody in the specimen. The result is positive or reactive.



Invalid: If no C-line develops, the assay is invalid regardless of any color development on the T-line as indicated below. Repeat the assay with a new test device.

THANKS

Add: 3rd floor, Xinhualian science and technology building, No.5

Jiachuang Road, Tongzhou District, Beijing, China

Tel: (8610) 69502385 Fax: (8610) 69507615

Whatsapp/Mob: +86-18242749295 (JAX)

Web: www.diagreat.com

Email: jax.chang@diagreat.com